AL-Technology

High Technology Implants

Catalog 2015/2016

Layout & Druck:

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I. The Implant System

Business firm	
The special Implant System	

II. Produktprogramm Implantate

Shark-Implantat, Drill-verfahren	8+9
Cylinder-Implantat, Drill-verfahren	10+11
Safe-Implantat, Drill-verfahren	12+13
Smart-Implantat, Drill-verfahren	14

II. Product program Implants

Shark-implant, Drill-procedure	
Cylinder-Implant, Drill-procedure	
Safe-Implant, Drill-procedure	
Smart-Implant, Drill-procedure	

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III. Surgery

Instructions	
Color Code	
Drills	
The intelligent drilling system in the world	
Surgical Kit	

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IV. Prothetic

Prothetic Accessories	
Clinical cases	

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V. Appendix

Price List	
Order Sheet	
Notice	
General terms and conditions	

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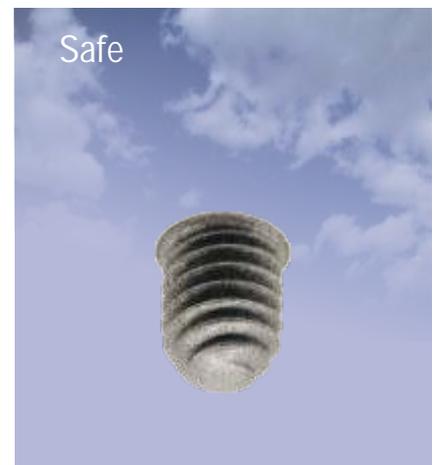
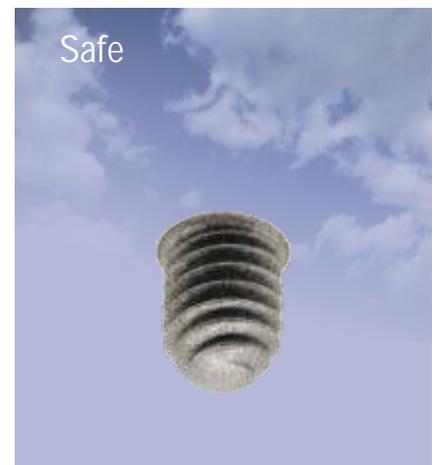
AL-Technology: Solution focused Quality conscious Certified quality

AL-Technology was founded in 2001 as an association of mechanical engineers, dental technicians, experienced dentists and oral surgeons. The result of this combination of experience and knowledge are excellent dental products, particularly in the field of implantology.

AL-Technology manufactured on modern and special CNC-controlled machines. The special shape of the implants, especially the conical offers easy implantation, maximum mechanical strength, stability immediately after implantation, and high bone integration.

The special treatment of the surface ((SLA: sand blasted, Large grit, Acid-etched), the special packaging of the implants in glass vials without plastics, in a sterile environment, guaranteed AL-Technology implants of excellent quality and the highest success rate. Our

customers in Germany, France and Spain are highly satisfied over the years with these quality standards. This high quality characteristics have been proved by several scientific and clinical studies.

Implants material: Titanium alloy Ti-6Al-4V ELI, Grade-5. AL-Technology Dental Devices implants and abutments are made of titanium alloy Ti-6Al-4V ELI.

High Technology Implants

Our company meets the international Standards and Certified by:

Authorized Representative In Europe:
MEDES LIMITED
5 Beaumont Gate, Shenley Hill, Radlett,
Herts WD7 7AR. England

Avanti guarantees the supply of implants and accessories for the next 20 years until 2030.

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Stability of conical design Uniform distribution of occlusal forces through internal hex connection
Ideal integration with the bone through SLA surface treatment

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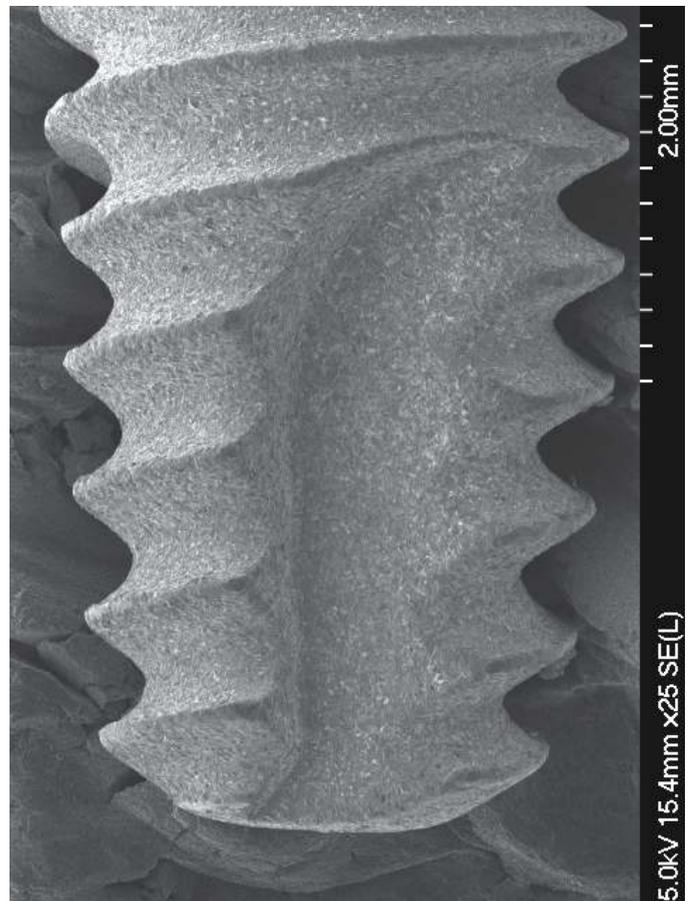
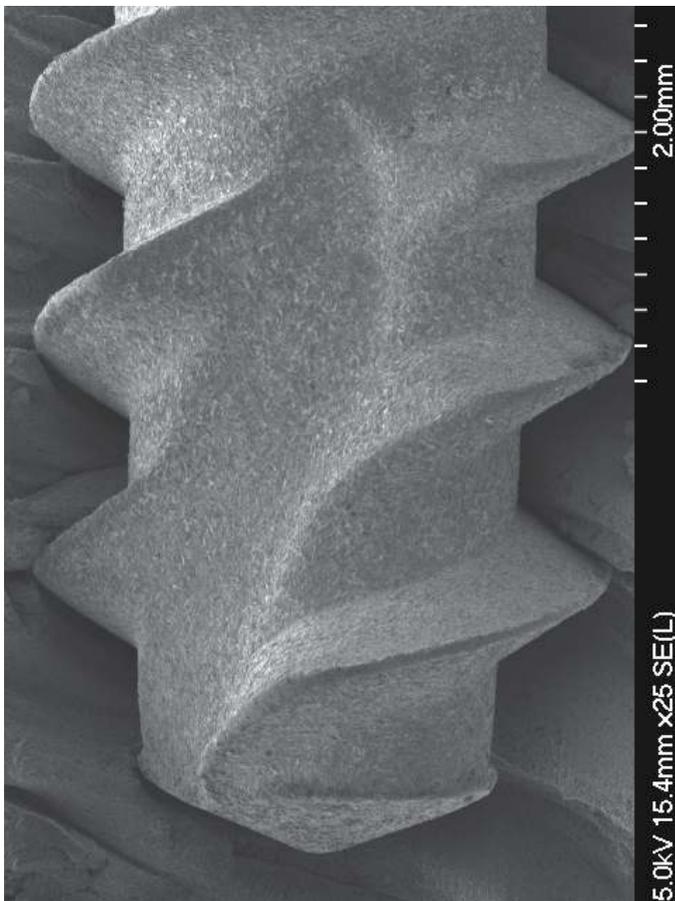
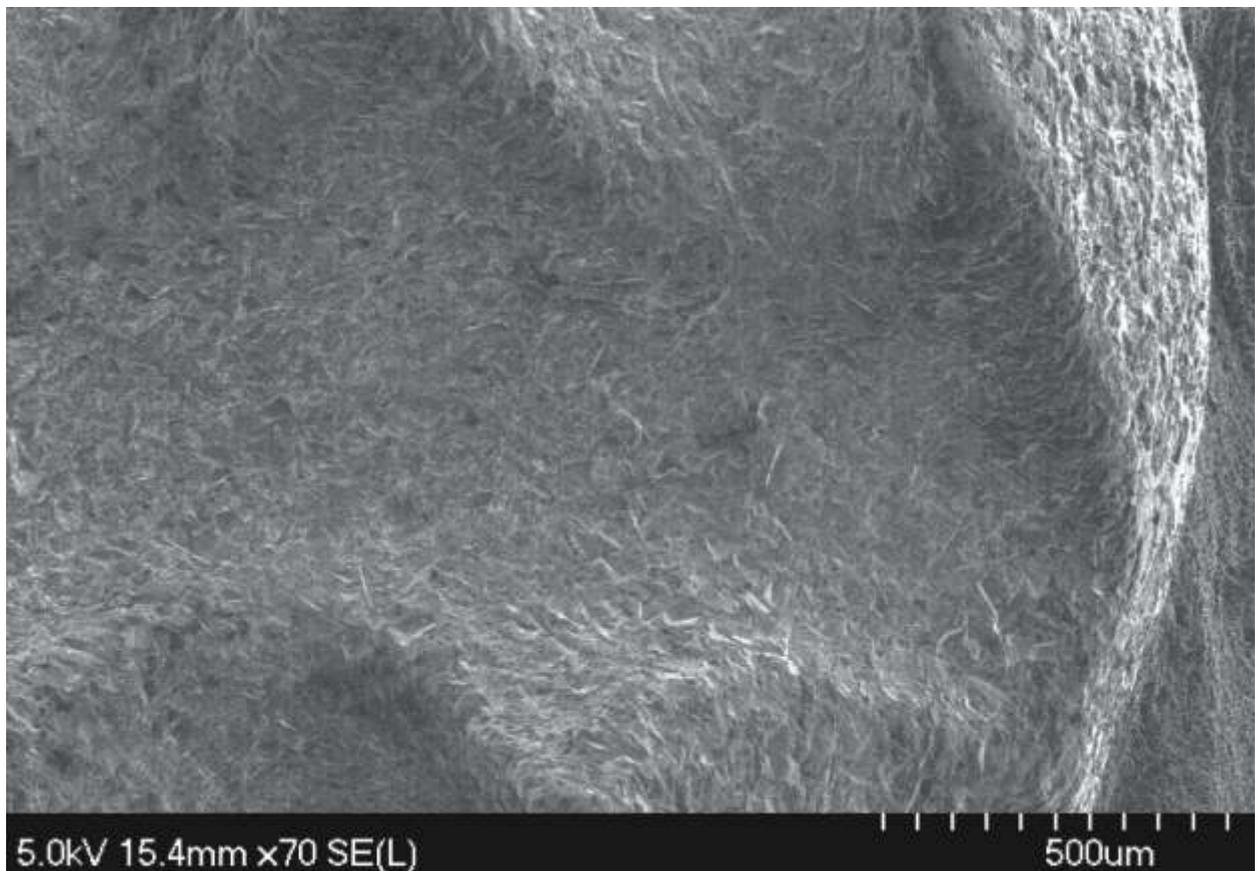
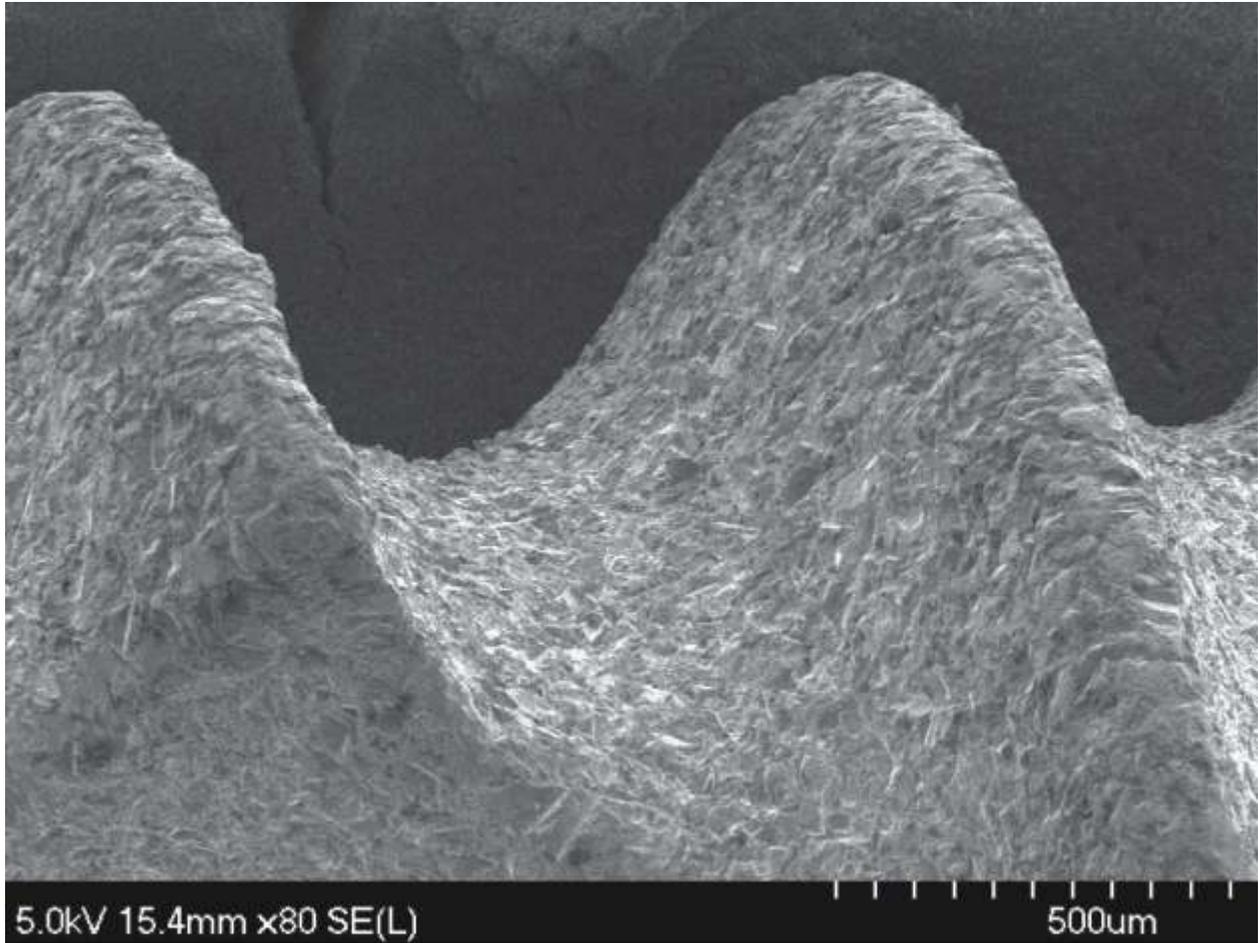
SHARK IMPLANTS with deep thread design.
It's designet for easy penetration, high stability and immediate loading

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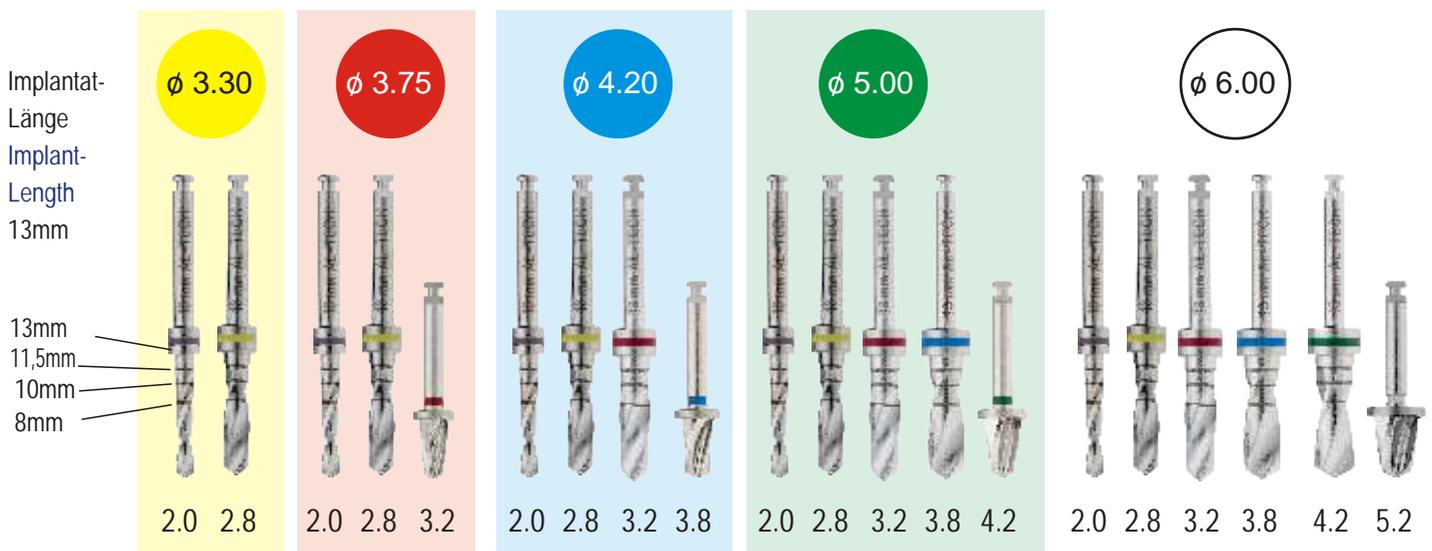
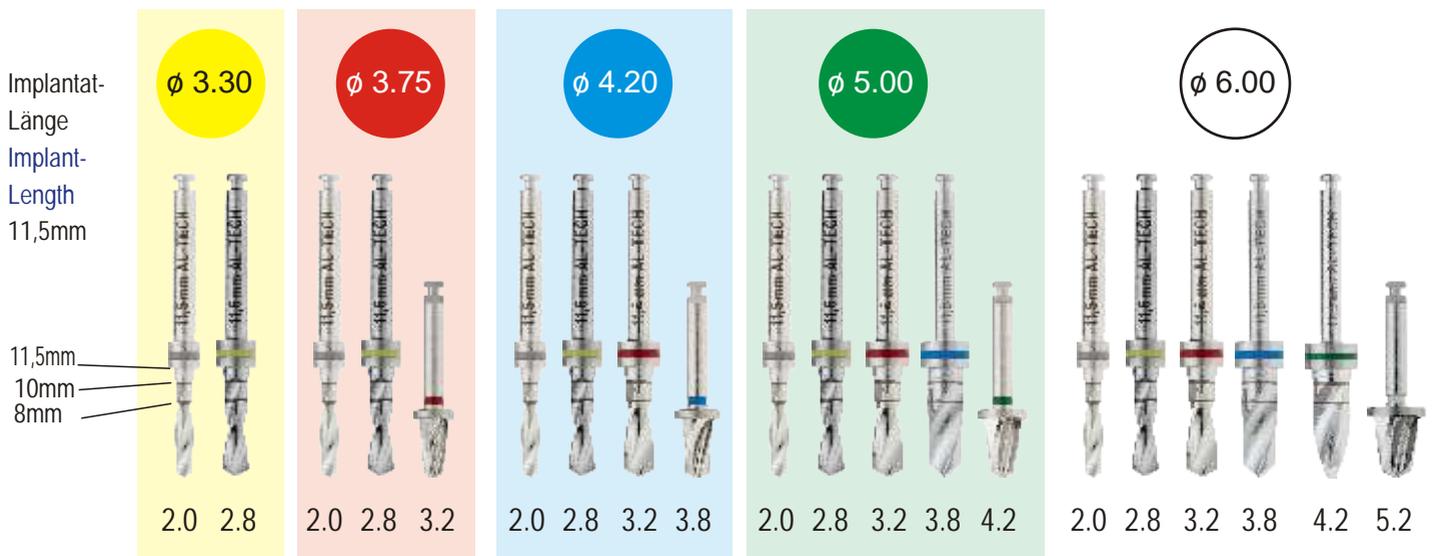
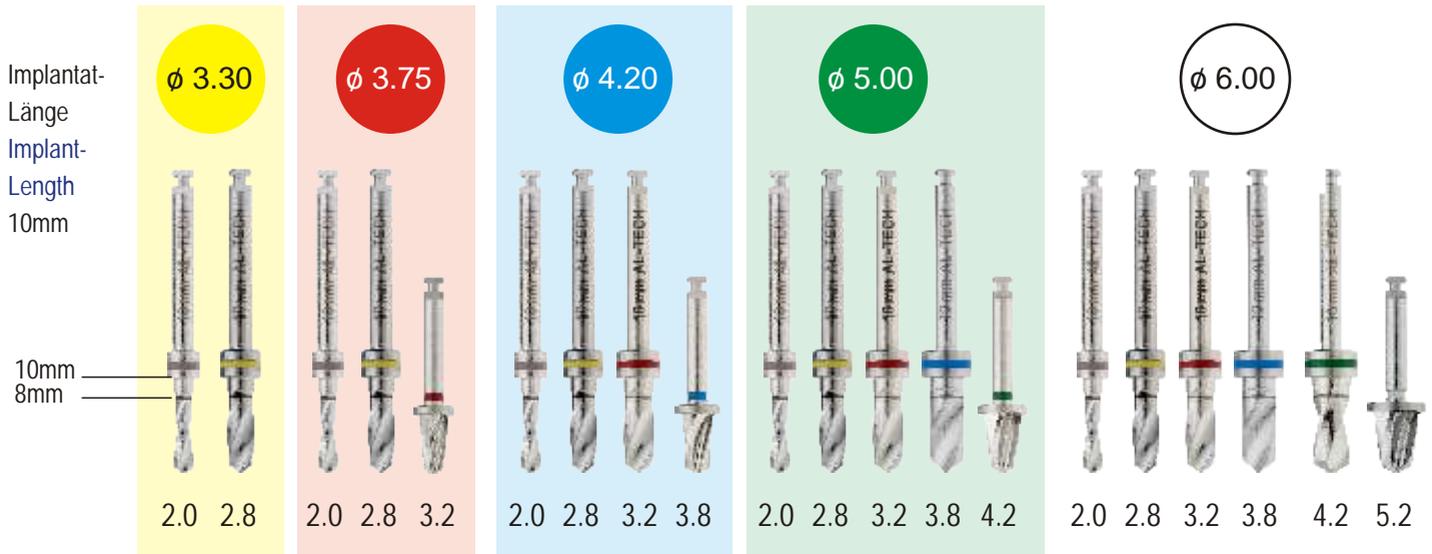
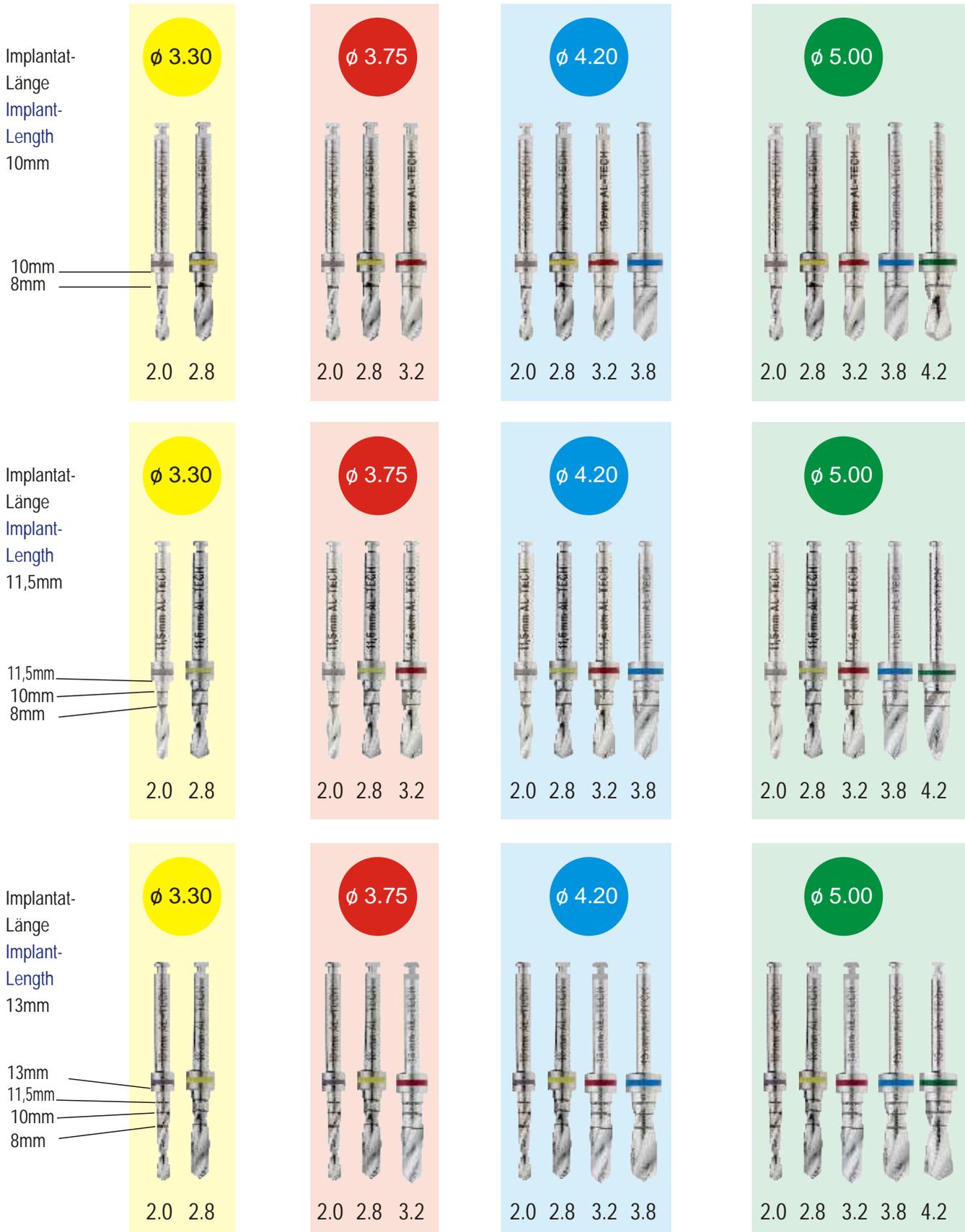
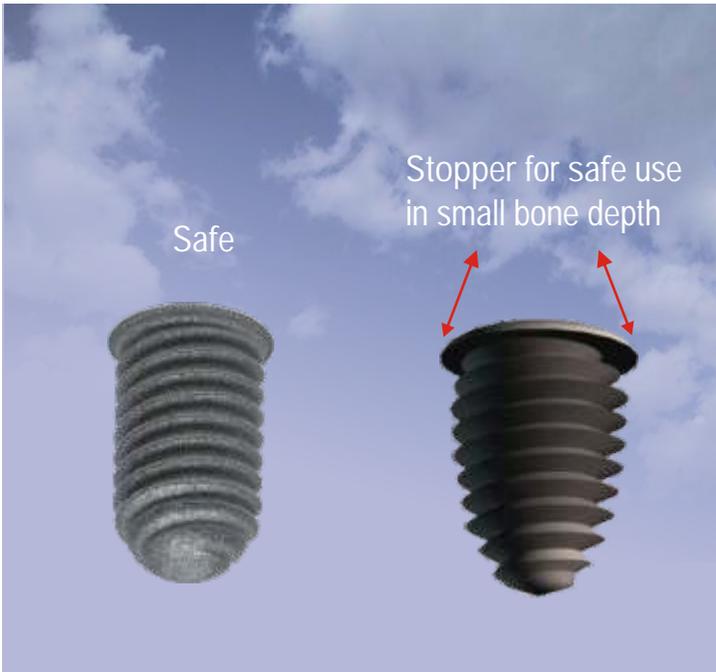
Safe Implants: a special design with fine deep thread and Stopper for safe use in small bone depth.

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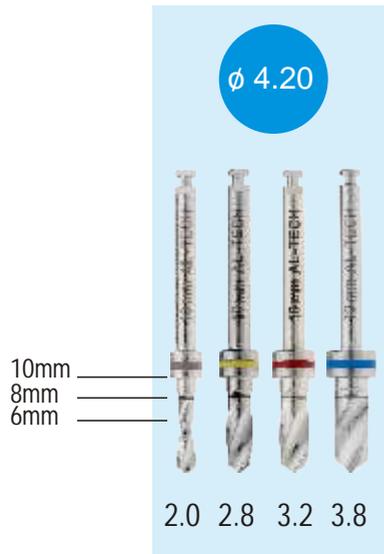
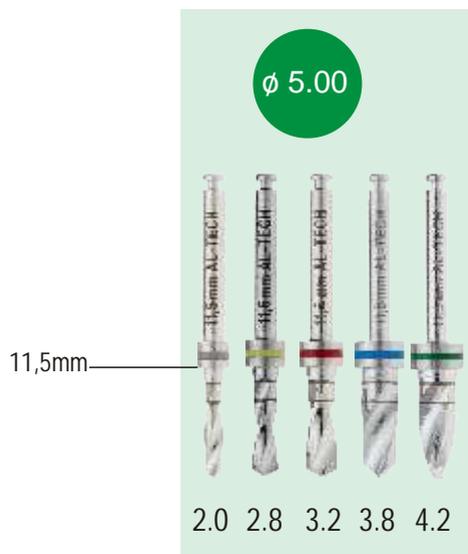
Smart Implants: Einteiliges Implantatsystem / One piece Implant System

Smart Implants:

Smart (One piece) implants designed for immediate loading where there is not enough bone in width drill should be easy 2m"m drill and implant is designed for easy penetration.

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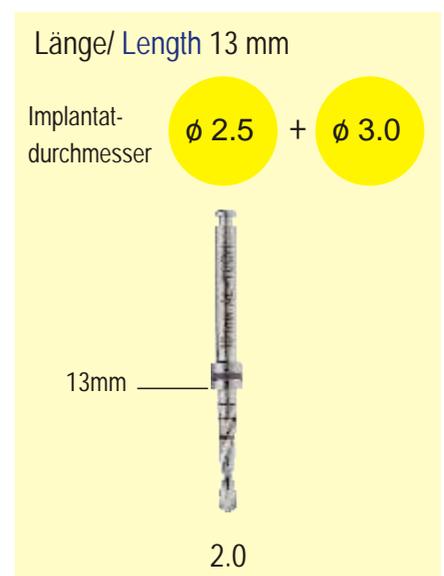
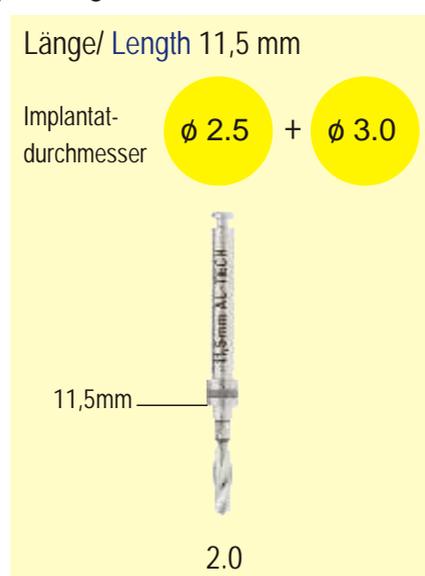
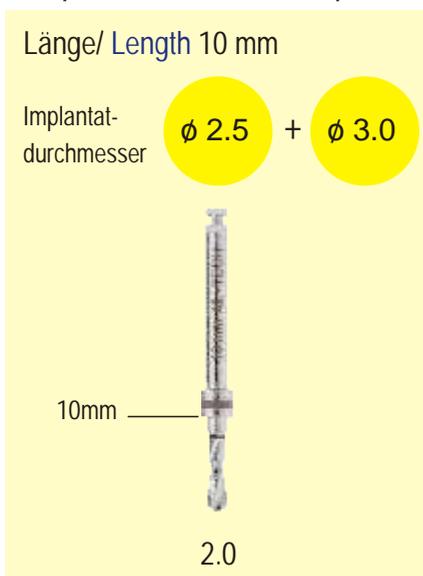
Instructions for use for AL-Technology dental
implants:

Warning: AL-Technology implants are products to be used only by certified dentists.

Symbols:

	See instructions	STERILE R	
	For single use only		Method of sterilization: irradiation
	Expiry Date		
	0473 CE marking		

Product packaging:

All implants are cleaned and packaged in double vials in special clean environment, Sterility guaranteed if pack unopened or undamaged, in case that the package was damaged or opened, do not sterile in any way!, in this case, implant should be returned to supplier for replacement, notice the expiration date on the pack before use, and implants should be used immediately after opening the vials.

Surgical planning and procedures:

- 1- To determine the adequate bone availability appropriate x-ray films should be used.
- 2- Choose the appropriate implants to be placed.
- 3- Choose the Drill procedure according to Implant type (Shark, Cylinder, Tornado, Safe)
- 4- Remove implant pack, and paste up the label with the data for each implant on the patient chart.
- 5- Open the outer vial.
- 6- Pull the implant by the insertion mount from the inner vial manually (s. figure), and place it immediately, rotate clockwise manually, then use a ratchet (1/4 inch)
- 7- Pull the insertion mount out of the implant.
- 8- Remove the cover screw on the top of the insertion mount (fig.-01), and insert it clockwise on top of the implant. (Use hex driver 0.05").

Figure :Components Al-Technology Implant type: Shark, Tornado, Cylinder, Safe (Internal hex 2.42 m"m):

Sterility guaranteed if pack unopened and undamaged, in case that the package was damaged or opened, do not use in any way!.

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The intelligent drilling system in the world

No more drill stopper and drill cores

Just take the drill that fits the implant length

The AL-Techno system offers a safe and easy drilling by the three drill length 10 mm, 11.5 mm and 13 mm to match the implant length

Take the Matching drill length and drill to the stop, easier and safer it gets.

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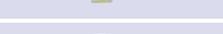
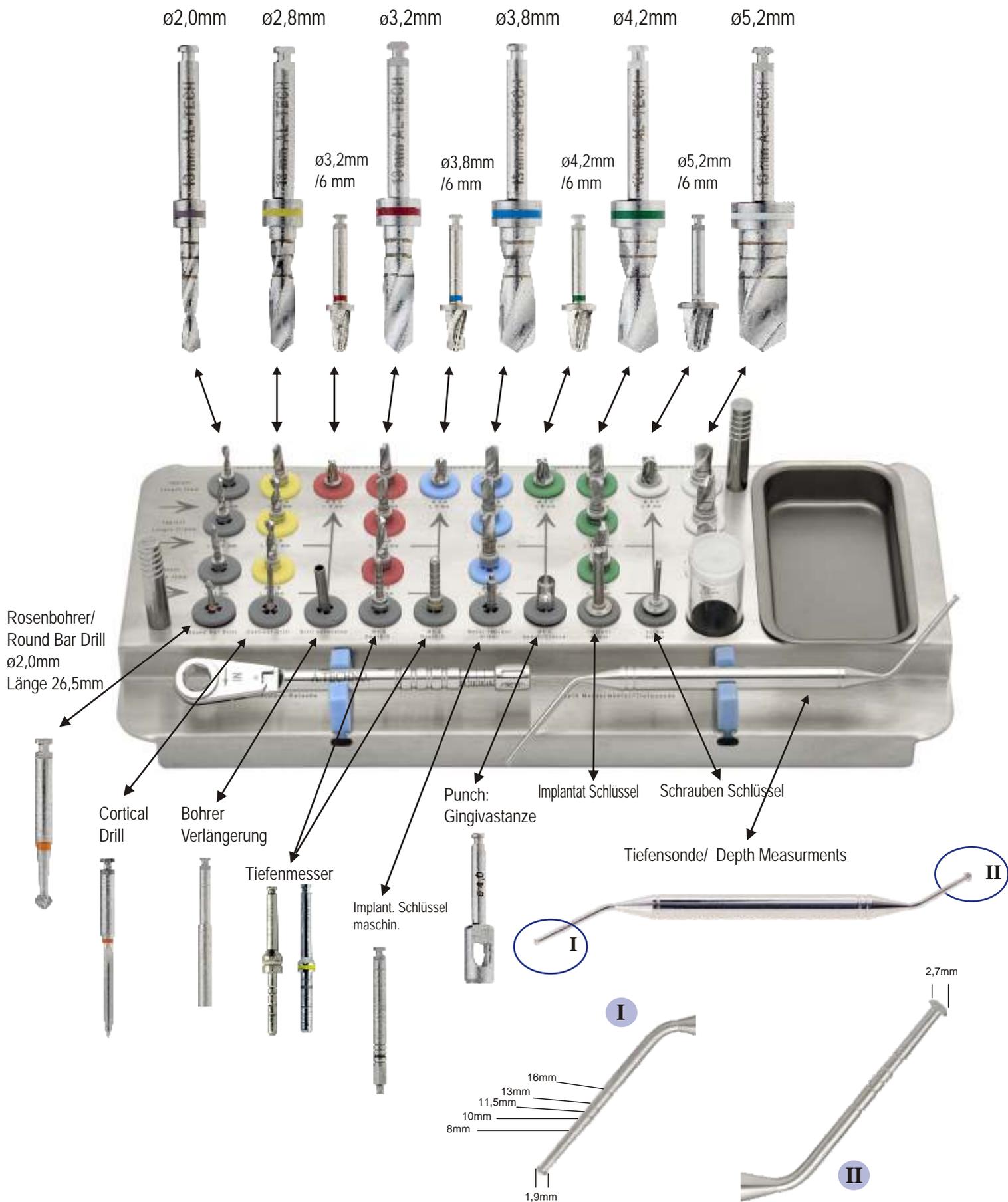
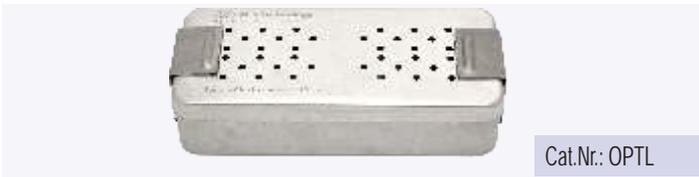
Cat.Nr.: OPTL

Implantatschlüssel maschinell
Motor Implant Driver

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Cat.Nr.: CER3

Implantatschlüssel lang / 18mm
Implant Driver Hex long / 18mm

Cat.Nr.: IMS

Tiefenmesser /Depth Gauge
2,0mm und 2,8mm

Cat.Nr.: TM20

Cat.Nr.: TM28

Einbringschlüssel /Prothetic Driver
1,25mm

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Avanti Technology GmbH
Lochhamer Schlag 11 a • 82166 Graefelfing/ Germany

Terms and conditions of sale and delivery

1. General

These terms and conditions of delivery apply for all of the products we sell. On granting the order the contractual partner acknowledges our terms and conditions. Deviating terms and conditions shall only apply if they are agreed in writing.

2. Conclusion, content of the contract

A contract shall only come into effect subject to our written order confirmation or on delivery of the agreed supplies/services. The written form shall also apply for assurances, ancillary agreements and subsequent amendments. The purchaser is bound by the granted order. The customer declares his agreement that we will check his credit worthiness. In the event of a negative result, we reserve the right to refuse to fulfil the order.

3. Prices and payment

The prices represent the value of goods and services without discounts and do not include the respectively applicable value-added tax. When a new catalogue appears, all previous price lists become invalid.

Our invoices are due for payment immediately and within 14 days of the invoice date insofar as nothing else is agreed.

4. Transfer of risk, inspection, obligation to make a complaint

The risk passes to the customer as of the goods being handed over to the party responsible for dispatch. Only on special written application by the purchaser will we have an insurance policy concluded at the expense of the purchaser. The contractual partner is obliged to inspect the goods for defects and to report any defects immediately, but at the latest within 14 days, in writing.

5. Guarantees, right of return

Insofar as nothing else is agreed in the following we shall guarantee the products in such a way that we will replace any defective products by products in perfect condition. The statutory limitation of the claim to a guarantee is 2 months as of the date of the transfer of risk.

The purchaser is obliged to inform us immediately on receipt of any material, supply or manufacturing faults as well as transport damage.

For consumables the purchaser has a right of return within 14 days for goods supplied by us.

6. Exclusion of compensation, liability limitation, withdrawal

If delivery cannot be made due to a fault on our part, the customer is entitled to withdraw from the supply contract or to demand compensation. The claim for compensation is limited to 5 % of the value of the goods ordered.

In the case of slightly negligent infringement of substantial contractual obligations, our liability for compensation is limited to the foreseeable damages typical for the contract.

In the case of slightly negligent infringement of non-substantial ancillary obligations, we accept no liability for payment of compensation. In all cases of liability for compensation based on a negligent infringement of an obligation, regardless of the legal basis, our liability shall be limited to the damages foreseeable for us.

Further compensation claims by the customer, regardless of the legal basis, are excluded. This does not apply to the extent that there is a necessary liability, for example, in the case of personal damages in accordance with the product liability law, or in cases of intent, gross negligence or the absence of assured properties.

If a claim is made against us for manufacturer's liability pursuant to § 823 BGB (claim in tort), our liability is limited beyond the preceding provisions to the compensation paid by our liability insurer.

Insofar as the insurance policy does not apply, or does not apply fully, our liability remains unchanged and limited to the sum insured. If the sum insured is not typical for the damages, the contract or the materials, our liability in such cases is limited to the amount typical for the damages, contract and materials. The preceding provisions do not apply in the case of damages to life, body or health and/or claims pursuant to the product liability law.

The statutory rights of withdrawal from the contract shall remain unaffected.

7. Reservation of ownership

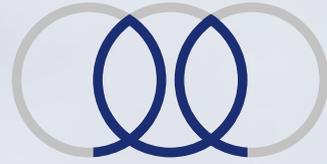
We reserve the right of ownership of the supplied goods until receipt of the goods value from the respective delivery contract. If the agreed payments are not made, we are entitled to demand the immediate return of the reserved goods.

8. Place of jurisdiction: Place of jurisdiction for all disputes arising from or in connection with this contract is Munich.

9. Ancillary agreements

Ancillary agreements shall be valid in writing only. In the event that individual provisions are ineffective, this shall not prejudice the validity of the other provisions. The ineffective provision shall be replaced by a provision which comes as close as possible to the ineffective provision.

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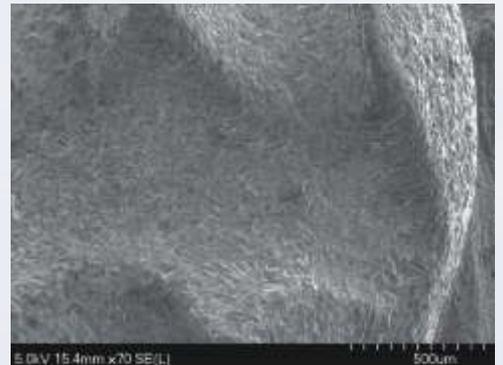
Stability of conical design
Uniform distribution of occlusal forces
through internal hex connection
Ideal integration with the bone through
SLA surface treatment

AL-Technology

High Technology Implants

Vertrieb durch: *Sales by:*
Avanti Technology GmbH
Lochhamer Schlag 11 a
D- 82166 Graefelfing/ Germany

Tel: 0049-89- 7474 7760
Fax: 0049-89-7675 8871

www.at-implantate.de

Mail: at-implantate@t-online.de

HRB: 132734 • USt-Nummer: 143/117/00058

Bank: Hypovereinsbank

BLZ: 70020270 • Konto: 33867670

Bank: Hypovereinsbank • SWIFT HYVEDEMMXXX
IBAN De82700202700033867670

www.at-implantate.de